

Product record

Initial

STK 12M

Maintenance

Packing list:

Clean Air IFU

Clean Air Step-by-Step Guide

Clean Air Plug-in Gauge

Clean Air

Model: CA-01 CA-02 CA-04

S/N: _____

Version: _____

STK / Maintenance must be performed at least every 12 months according to the manufacturer's information given in the product's Instructions for Use (IFU) or in the manufacturer's STK /Maintenance instruction. A successful STK /Maintenance requires all steps of this record to be successfully performed and all tests rated PASS. In case of a test FAIL call service.

A. Connection test	<input type="checkbox"/> Performed	<input type="checkbox"/> Not performed
<ul style="list-style-type: none">• Check connection of Clean Air Plug-In Gauge• Check connection of Clean Air Vacuum tube	Result	<input type="checkbox"/> PASS <input type="checkbox"/> Fail
<hr/>		
B. Visual test	<input type="checkbox"/> Performed	<input type="checkbox"/> Not performed
<ul style="list-style-type: none">• Check safety relevant labeling on the device for availability and readability.• Check mechanical parts are not defective.• Check for damage and debris.• Check relevant accessories: Plug-In Gauge	Result	<input type="checkbox"/> PASS <input type="checkbox"/> Fail
<hr/>		
C. Functional test	<input type="checkbox"/> Performed	<input type="checkbox"/> Not performed
<ul style="list-style-type: none">• Operation-Test: During operation the Clean Air Plug-In Gauge shows the green range• Noise level test Noise \leq 44 dB (A)	Result	<input type="checkbox"/> PASS <input type="checkbox"/> Fail

With my below signature I confirm having performed the above tests and ratings correctly.

Date

Signature

Company's stamp

